

Blendex™

Automatic Alginate Mixer



ISO 9001
ISO 13485



Blendex™

- Improved special gear design is more stable and durable with less noise and features built-in thermal overload protection.
- Automatic trouble-free alginate mixing at the touch of a button.
- Time saving process takes only seconds to complete.
- Consistent, smooth and bubble-free mixing each and every time.
- Features automatic timer with audible tone.
- 3 customizable memory presets available.
- LED display shows mixing time and memory presets.
- Automatic door safety lock ensures safe operation.
- Unit is available in AC100V 50/60Hz, AC110V 50/60Hz, or AC230V 50/60Hz operation.

DENTAMERICA®

Blendex™

Automatic Alginate Mixer



1 Add alginate and water to the mixing cup.




2 Place the mixing cup inside well.



3 Press the "START" button for a perfectly smooth Blendex™ mix.


The Blendex system is equipped with 3 pre-set timing modes (these settings can be changed.)

Memory	Mixing Time (sec)
☺	8
☺☺	10
☺☺☺	12

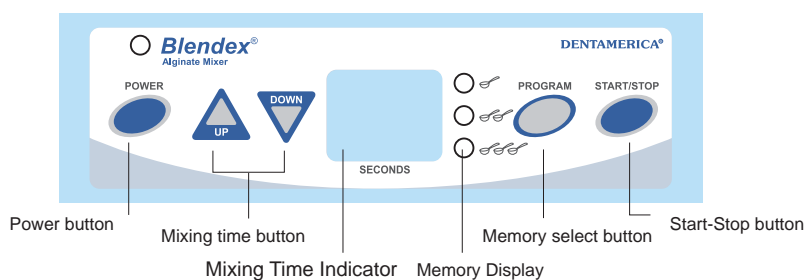

SPECIFICATIONS

Dimensions	W205 X D245 X H285 (mm)
Weight	19.5 kg (±0.5kg)
Voltage	AC230V (50/60 Hz) AC110V (50/60 Hz)
Maximum Power	Maximum 350VA
Timer	1-99 sec, adjustable by time
Memory	3 (installed settings: 8, 10, 12 sec)
RPM	2,900 RPM

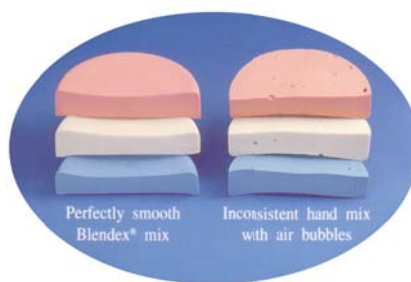

DENTAMERICA®

18688 E. San Jose Ave.
City of Industry, CA 91748 U.S.A.
Phone: (626) 912-1388
Fax: (626) 913-0510
www.dentamerica.com
www.dentamerica.asia

DISTRIBUTOR

DENTAMERICA®